

# Disposable Face Mask

## Features

- Adjustable nose piece
- Elastic earloop
- Minimal protection non-medical
- Not for medical use



## Specification

Colour	Blue
Size	175 x 95 mm
Material / Fabric	3 ply, Non-Woven 1. ply: Spun Bond PP 2. ply: Melt Bond PP 3. ply: Spun Bond PP
Expire Date	2 Years

### Appropriate use of face masks is important

The face mask should completely cover the face from the bridge of the nose down to the chin. Clean hands with soap and water or alcohol-based hand sanitiser before putting on and taking off the face mask. When taking off the face mask, remove it from behind, avoiding to touch the front side. Dispose the face mask safely if it is disposable. Wash your hands or apply alcohol-based hand sanitiser immediately after removing the face mask

## Instructions

1. Hold mask on face and pull the elastic band to ears
2. Press the nose clip to fit the shape of the nose
3. Pull and extend the mask with two hands making sure the nose clip is in an upward position

## Precautions

1. The mask should be replaced over time and is not recommended for long-term use
2. It is recommended to stop using the mask if there are any adverse reactions during wearing
3. This product is not washable, please make sure it is used within the validity period
4. Store in a dry and well-ventilated area away from fire and flammable objects

## Storage

Please store it in relative humidity of less than 80%, in non-corrosive gas and in a well-ventilated area, away from a high temperature environment

**Art. Nr.**  
**RND 600-00234**